

Title: Designated Suction tubing required for Urinary Catheter

WARNING: Suction Tubing & Purewick, Female Urinary Catheter

Date: May 2017

SP	Ang, Steven	ML	Leonor, Nicholas
AKS	Bandian, Leo	E.L. (M)	Lewis, Eula
db	Barnoski, Ingrid	SM	Mahboobi, Safi
MB	Bell, Malika	EM	Mangalindan, Earl
MB	Booze, Joseph	JL	Martin, Laura
MB	Burns, Rachael	AM	Matsushima, Aya
GRS	Burns, Greg	CM	Milleza, Consuelo
FC	Calma, Franco		Mojadeddi, Moman
LC	Carothers, Logan	CW	Moon, Cheryl
SC	Castillo, Sam	JN	Nastor, Jessica
CC	Cespedes, Clara	JMK	Navarro, Anggie
JL	Chow, Joanna	MU	Nelson, Mike
MD	Decuire, Byron	Hs	Ong, Hannah
MD	Diaz, Martha	LP	Pangilinan, Lance
RE	Ennis, Robert	RP	Perez, Ricardo
QED	Fazel Darbandi, Jill		Phillips, Justin
H	Fejerang, Rosemarie	AG	Quong, Andrew
MJA	Ferrer, Jason	CH	Rahming, Chanera
SP	Fisher, Paul	DS	Saladze, Dimitri
N/A	Fong, Myron	ES	Saliba, Edward
LOA	Foster, Loretta	SBS	Schletz, Susan
ly	Gatchalian, Leslie	MS	Scholz, Misty
HH	Hammoudeh, Hanaa	TS	Sikhounchanh, Annie
KH	Ho, Pik Wah (Kelly)		Summers, Travis
JH	Hsieh, Jenny	LOA JS	Sylvester, Josephine
JJ	Johal, Jasmeet	W	Vides, Andrew
	Kallet, Richard?	RETIRED	Vinson, David
	Kelly, John	SV	Waugh, Shane
	Lear, Veronica	OTW	Wedeking, Grey

Title: Designated Suction tubing required for Urinary Cathe

WARNING: Suction Tubing & Purewick, Female Urinary Catheter

Date: May 2017

DW
VY

Whiting, Douglas

Yip, Vivian

PureWick® External Catheter for Women



Printer-friendly version




PureWick® External Catheter for Women provides an effective alternative to internal catheters and a sanitary solution for patients who have difficulty transferring to the toilet. The soft, flexible wick connects to a wall vacuum system or a DryDoc Vacuum Station to wick urine away. PureWick® keeps users dry and odor free while minimizing or eliminating catheter associated urinary tract infections (CAUTI), pressure sores and moisture associated skin damage

Thank you for your interest in PureWick® External Catheter for Women.

PureWick is an external catheter for bed- and wheelchair-bound women that provides a safe, efficient and comfortable alternative to Foley and suprapubic catheters. Using PureWick can minimize or eliminate catheter associated urinary tract infections (CAUTI), pressure injury, and moisture associated Skin Damage (MASD). Hospitals are seeing significant improvement in CAUTI and HAPU rates after implementing PureWick.

The soft, flexible and disposable 'wick' tucks snugly between the labia and gluteus muscles and is connected to gentle, continuous-pressure vacuum. **In hospitals, the wick connects via tubing to the wall vacuum system requiring only the wick be purchased.** Home and facility users can purchase or lease our compact DryDoc Vacuum Station™.

One of the best features of PureWick is its simplicity and ease of use. The 3-minute In-Service training video included on the USB provides the necessary information for successful implementation. In addition, our Clinical Specialist, Karen Enciso, is available by phone (760) 522-6925 or email kenciso@purewick.com for further support.

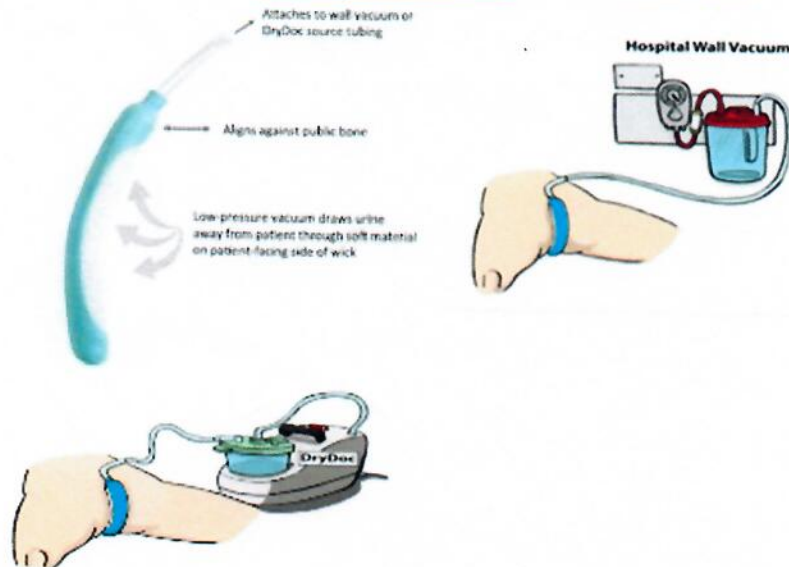

PureWick is a small business, woman-owned, Gulf War veteran-owned company and received the 2015 'Most Innovative Product' award (Medical Device and Pharmaceutical Category) from San Diego CONNECT.

I have attached our product brochure, clinical/economic benefits, white paper and additional documents as requested. Please visit www.purewick.com to view a video on the "How it Works" page: <http://www.purewick.com/how-does-it-work-2/>.

Please feel free to contact me directly with any additional questions or to schedule a 30 wick no cost trial.